AllCheck COVID19 Ag

Einstufiger COVID19 Antigen-Test

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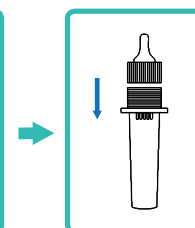
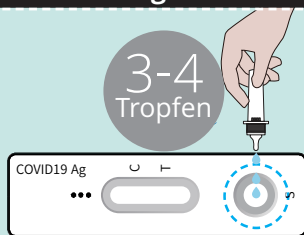
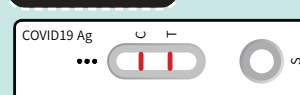
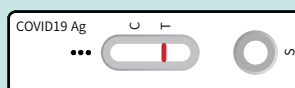
AllCheck COVID19 Ag

One step COVID19 Antigen Test

All Check COVID19 Ag is a rapid immunochromatographic test for the qualitative detection of specific antigens to SARS-CoV-2 present in human nasopharynx.

May also be useful for supporting public health strategies, such as contact tracing and largescale testing of people suspected of having an active infection.

Features

- ✓ Easy to use and Fast test time: 10~15minutes
- ✓ All necessary reagents provided
- ✓ Suitable for Point-of-Care testing
- ✓ No need for extra equipment

Reagents and Materials

Test Device, Extraction buffer tube, Nozzle cap, Sterile swab

Ordering Information

Cat.No	Type	Pack Size	Specimen	Sample type	Shelf Life
CHR11	Device	20T/Box	Nasopharyngeal secretion	1~30°C (33.8~86°F)	18months

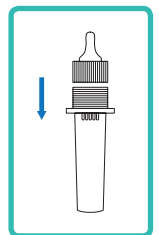
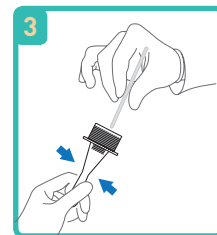
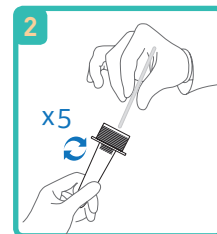
AllCheck COVID19 Ag

One step COVID19 Antigen Test


Collection of specimen [Nasopharyngeal swab]

1. Insert a sterile swab into the nostril of the patient, swab over the surface of the posterior nasopharynx. Withdraw the sterile swab from the nasal cavity.
2. Insert the swab into an extraction buffer tube. While squeezing the buffer tube, stir the swab more than 5 times.
3. Remove the swab while squeezing the sides of the tube to extract the liquid from the swab. Press the nozzle cap tightly onto the tube.
4. Apply 3~4drops of extracted specimen to the specimen well of the test device.

Test Procedure

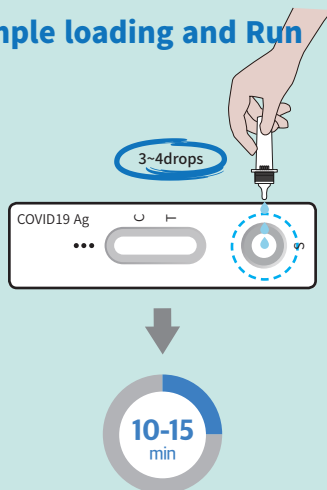Sample loading and Run


Result Interpretation


Negative

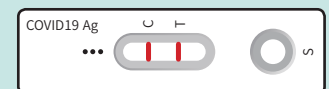Positive

Invalid
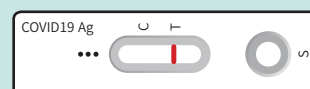
<http://www.thecalth.com> / info@thecalth.com

Headquarters : #321, 54, Changeop-ro, Sujeong-gu, Seongnam-si, Gyeonggi-do, 13449, Republic of Korea
Tel : 82-31-754-0320 Fax : 82-31-754-0321

Manufacturing site : #7508, 140, Beolmal-ro, Dongan-gu, Anyang-si, Gyeonggi-do, 14057, Republic of Korea
Tel : 82-31-360-0328 Fax : 82-70-8228-0328